

# THE ESSENTIAL BUILDING BLOCK FOR YOUR GRAFT SITES

NuOss™



**NuOss™** Collagen is a natural bone substitute that supports bone growth in both periodontal and maxillofacial defects. **NuOss™** Collagen consists of Cancellous **NuOss™** particles which are bound together with approximately 5% collagen. When inserted (dry) the **NuOss™** Collagen hydrates in place, becoming moldable to adapt to the defect. **NuOss™** Collagen combines the mineral properties and space maintenance of **NuOss™** particulate with the wound healing properties of collagen. Due to binding of the **NuOss™** Collagen particles, minimal loose bone graft particles migrate out of the defect.

#### Indications For Use

- Filling of Extraction Sockets
- Filling of Peri-Implant Defects
- Filling of Periodontal Defects
- Augmentation of the Alveolar Ridge

[www.acesurgical.com](http://www.acesurgical.com)

ACE Surgical Supply Company, Inc. • 1034 Pearl Street, Brockton, MA 02301




NuOss™ Collagen Hydrated




**NuOss™  
Collagen**


## SOCKET PRESERVATION WITH NuOss™ COLLAGEN

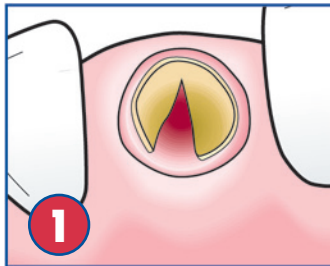


**1**  
After extraction, the socket shows a significant buccal wall bone dehiscence.

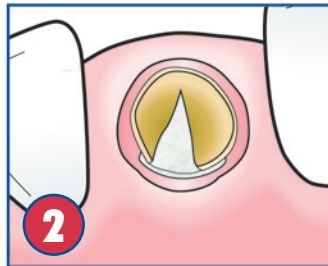


**2**  
Prepare a subperiosteal buccal pocket and place a long term resorbable collagen membrane (RCM<sup>®</sup>) to cover the defect.

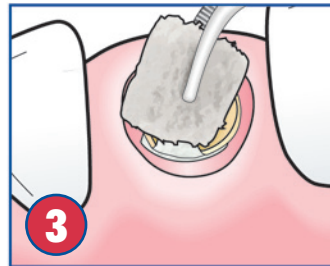


**3**  
Remove NuOss™ Collagen from sterile packaging, and place into bleeding socket. There is no need to hydrate the material beforehand.

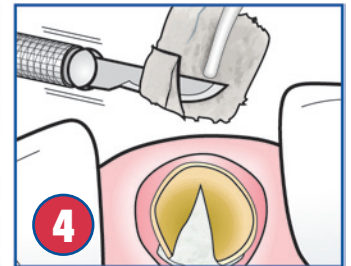


**4**  
If desired, NuOss™ Collagen may be cut to allow for easier placement.

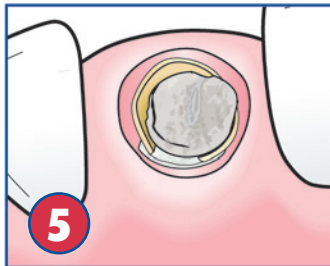


**5**  
Allow material to hydrate in the bleeding socket.

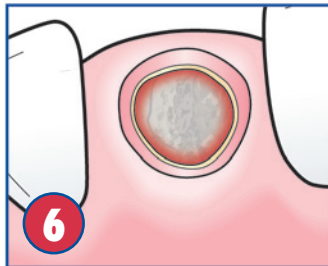


**6**  
NuOss™ Collagen will become moldable and pliable during hydration, allowing you to easily adapt the material to the defect.

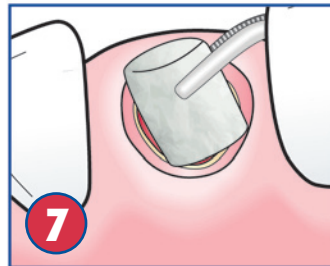


**7**  
Isolate the defect from the oral environment. Popular products include Resorbable Collagen Plug, Tape, or Foam.

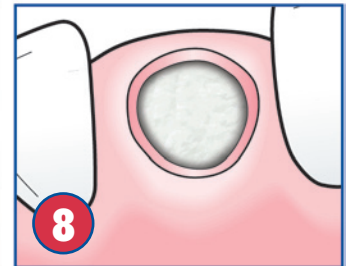

**8**  
For best results, cover the NuOss™ Collagen completely.

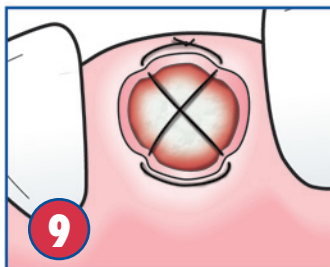

**9**  
Secure the collagen plug, tape, or foam with simple mattress sutures.

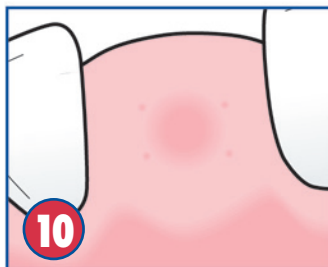

**10**  
Allow to heal completely.

### NuOss™ COLLAGEN Available sizes

Number	Block Size	Approx. Volume	Price
509-9100	6mm x 7mm x 8mm	100mg	\$79.99
509-9250	8mm x 9mm x 9mm	250mg	149.99
509-9500	10mm x 11mm x 12mm	500mg	249.99